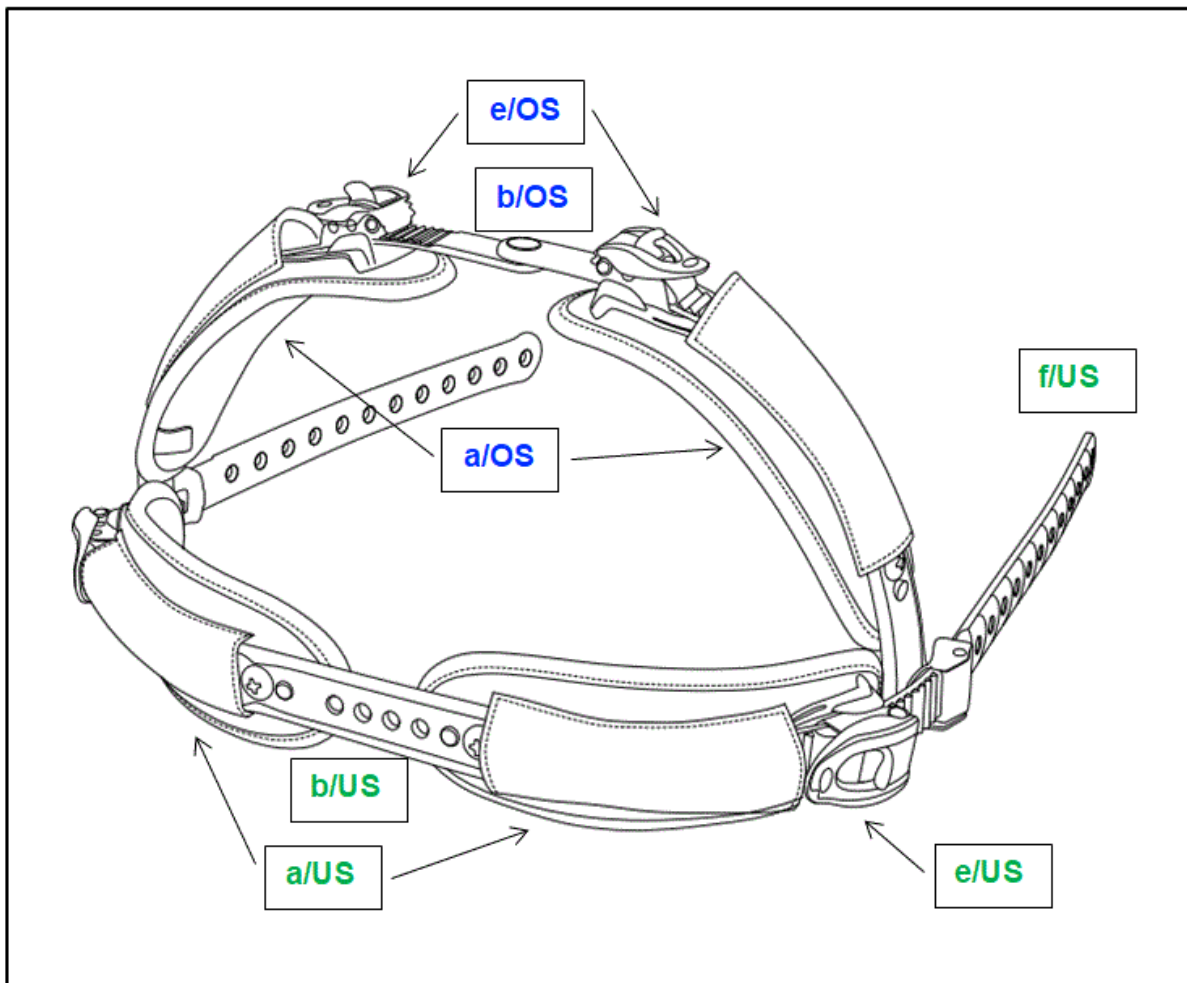




**PELVI.LOC®**

**Genuframe  
PL-GF - XXS - XS - S - M - L - XL**



For the safe use of this product, read and observe these assembly instructions and the manual before the first use.



### Intended use

The **PELVI.LOC®** PL-GF is intended for people with restricted mobility, especially for thigh positioning within the wheelchair and active-wheelchair and prevents from sliding forward.

### Intended Patient Group:

- Infants, Adolescents and adults
- gender-neutral

### Receipt Control

Completely unpack the product and check the delivery for completeness and transport damages. Should you – against expectation – find a defect in the delivered product then:

1. Immediately lodge a complaint with the freight carrier
2. Have the damage/defect recorded

3. Notify us or your retailer immediately

### Scope of Delivery

- 1) 1 x **PELVI.LOC®** PL-GF - XXS, -XS, - S, -M, -L, -XL pre-assembled
- 2) x **PELVI.LOC®** PL-GF - XXS, -XS, - S, -M, -L, -XL pre-assembled
- 3) 1 x Information for online download of the manual

### Combination of medical devices

Suitable for medical devices which are intended for the use of a Genuframe

### Fitting the lower leg strap



Installation should be carried out by a person with specialist knowledge

The PELVI.LOC® Genuframe lower leg strap is fitted in a similar way to the PL-UG

1. place the pads (**a/US**) in the lower leg area (below the head of the tibia or fibula). The hole/tooth band (**f/US**) is on the outside.
  - The PELVI.LOC® logo on the lower leg pad can then be read from the front!
2. adjust the distance between the lower leg pads using the perforated strap (**b/US**).
3. attach the hole/tooth strap (**f/US**) to the frame tube of the wheelchair
4. tighten the hole/tooth strap (**f/US**) equally on both sides until there is enough space for a flat hand between the thigh and the padding
5. if necessary, the ends of the toothed band can be cut to half-round length. If necessary, deburr the ends of the cut-to-length strap. -Risk of injury!
6. check the tight fit and the locking mechanism on both sides of the hole/toothed strap (**f/US**)!

### Fitting the thigh strap

The PELVI.LOC® Genuframe thigh belt is fitted in a similar way to the PL-2DB

7. place the pads (**a/OS**) in the thigh area. - The toothed strap (**b/OS**) is on top.
  - The PELVI.LOC® logo on the thigh pad can then be read from above!
8. adjust the distance between the pads over the toothed strap (**b/OS**) by operating the ratchet buckles of the thigh strap (**e/OS**).
  - The joint of the toothed strap (**b/OS**) should then be centered between the pads.
  - Pull the toothed strap out as far as possible so that two teeth are still free behind each ratchet buckle (**e/OS**).



## Manual

### Safety



### Notes

In spite of their stable construction, components like the hole straps, webbings, ratchets and paddings are subject to wear and tear. It is therefore recommended to perform a thorough inspection of the **PELVI.LOC®** every 3 Months. It is thereby important to make sure the rivet and screw connections (d) are firmly secure! (if applicable)

- Before using the product, ensure that it is in a serviceable condition and that it fits correctly. – Danger of accidents!
- Do not patch up and/or glue up and reuse damaged components such as hole strap (f), ratchet (b) or padding (a). – Danger of accidents!
- Do not sit on the paddings – danger of bruising, especially by the buckles (c)!
- Do not lift the wheelchair by the **PELVI.LOC®**. Danger of accidents!
- Be careful that your hands do not get caught between the hole strap and the frame or seat tube. Danger of crushing!

### Restrictions on use / undesirable side effects

- Do not use the **PELVI.LOC®** if there are any bruises, wounds or other not well-healed injuries in the area where the paddings will be. – Danger of injury!
- Do not use the **PELVI.LOC®** in case of osteoporosis, brittle bone disease or severe circulatory disorders. – Danger of injury!
- After cleaning, ensure the correct functioning of the **PELVI.LOC®**!
- A damaged **PELVI.LOC®** must be replaced immediately. – Danger of accidents!



### Attention:

- The **PELVI.LOC®** is not part of the restraint system for the wheelchair and/or user during transport in a vehicle for the transport of disabled persons.
- Neither the manufacturer nor the distributor will be liable, in the sense of product liability, for damages or risks caused by unattended children with regard to the product!

## Handling

1. to use the PELVI.LOC®, the pads (a) are placed on the thighs and lower legs



Caution: Make sure that there are no items under the padding! This will prevent painful bruises.

2. then guide the toothed strap (f/US) through the respective ratchet. To do this, press the corresponding ratchet buckle (e/US).
3. then tighten the toothed strap (f/US) until there is enough room for a flat hand between the thigh and the pad.

4. make sure that the ratchet buckles (e/US) engage in the teeth of the toothed strap!

➤ Carry out a tensile test!

To open the PELVI.LOC®, operate the ratchet buckle (e/US) and pull out the toothed strap.

➤ Make sure that the pads do not get between the spokes of the drive wheel!

## Cleaning and Disinfection

- Machine washable up to 40 degrees.
- Thermochemical disinfection possible by adding ELTRA 40 when washing
- No spinning
- Not dryer-proof

## Maintenance

Inspect the **PELVI.LOC®** for damages and check the functioning on a regular basis.

## Repair



Immediately replace defective **PELVI.LOC®** systems.

## Disposal

- The packaging is recyclable.
- The metal parts can be disposed of through scrap metal recycling.
- Plastic parts are suitable for plastic recycling.
- The disposal must be made according to national statutory provisions.
- Please inquire with your city or municipal administration about local disposal services.

## Technical Data

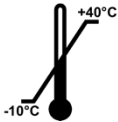

	XXXS	XXS	XS	S	M	L	XL
<b>Seat width (cm)</b>		12 - 20	16 - 24	20 - 32	30 - 40	36 - 46	44 - 52
<b>Padding size (cm)</b>		14 x 6	18 x 7	21 x 8	25,5 x 8	26,5 x 8,5	28,5 x 9
<b>Weight (g)</b>		220	220	220	280	280	280
<b>Predetermined breaking point (at approx. kg)</b>		40	60	60	80	80	80

<b>Ratchet size XXXS - XXS</b>	1,9 x 0,2 x 40 cm	<b>Product is latex-free</b>    
<b>Ratchet size XS - S</b>	1,9 x 0,2 x 40 cm	
<b>Ratchet size M - L - XL</b>	2,4 x 0,2 x 40 cm	
<b>Buckle size XXXS</b>		
<b>Buckle size XXS</b>	B-19-ERGO-BL	
<b>Buckle size XS - S</b>	B-19-ERGO-BL	
<b>Buckle size M - L - XL</b>	B-24-ERGO-BL	



**Important note: Knee width, not seat width!**

## Storage and transport conditions

	<p>Temperature range for long-term storage from -10°C to +40°C.</p> <p>Short-term temperature peaks, e.g. during transport, from -20°C to +60°C are well tolerated by the material and do not affect the safety and performance of the product.</p>
	<p>Protect from moisture during storage and transport</p>

## Warranty

We provide warranty for this product in accordance with the legal provisions. This warranty includes material and processing defects. This does not include wear parts and components/assemblies that are subject to intended wear and tear as well as damages that are the result of excessive strain, unintended use, forcible damage or unauthorised modification/repair. If you wish to make a warranty claim, please inquire with the retailer you purchased the product from. For product liability reasons, repairs and maintenance work must only be conducted by authorised speciality retailers, using original spare parts.



### Manufacturer:

A2J GmbH  
Am Fuchsberg 13  
D-87452 Altusried  
Fon: +49 (08373) 987269  
Fax: +49 (08373) 987276  
eMail: [info@a2j-intl.com](mailto:info@a2j-intl.com)  
Internet: [www.a2j-intl.com](http://www.a2j-intl.com)

As of 01.01.2025



This product complies with the essential requirements of the EU Regulation 2017/745 on medical devices – Medical Device Regulation (MDR)

Subject to technical modifications