



PELVI.LOC® PL-KG-GO - S - M - L - XL



For the safe use of this product, please read and follow these assembly and usage instructions before first use



Area of application

The **PELVI.LOC®** PL-KG-GO is specially designed for people with limited mobility to position their legs on the footrest of the wheelchair and prevent them from sliding forwards.

Intended patient group:

- Children, adolescents, and adults
- Gender-neutral

Receipt control

Unpack the product completely and check the shipment for completeness and transport damage. If you unexpectedly find any defects in the shipment:

1. Immediately report the issue to the carrier
2. Have the damage or defect documented
3. Inform us or your specialist dealer without delay

Scope of delivery

- 1) 1 x **PELVI.LOC®** PL-KG-GO ankle gaiter -S, -M, -L, XL pre-assembled (pair)
- 2) 1 x Mounting set
- 3) 1 x Information for online download of the assembly and usage instructions

Combination with medical devices

Suitable for aids that are intended for use with ankle gaiters

Mounting ankle gaiters with ratchet buckles



Assembly should be carried out by a person with appropriate expertise

1. Attach the two straps of the ankle gaiter to the footboard using the mounting set.
2. Place the pads (a) in the ankle area. - The toothed strap (b) is at the front.
 - The logo on the padding can be read from the side!
3. Adjust the distance between the pads over the toothed strap (b) by operating the ratchet buckles (e).
 - The toothed strap should then be in the centre front, between the pads.
4. If necessary, the ends of the toothed strap can be trimmed to a rounded shape. Deburr the trimmed ends of the toothed strap if needed – risk of injury!
5. Check the secure fit and locking on both sides of the toothed strap!



Instructions for use



Safety

instructions

Despite its robust construction, components such as belts, toothed straps, and padding are subject to wear and tear. Therefore, it is recommended to thoroughly inspect the **PELVI.LOC®** every 3 months. Ensure that rivet and screw connections are securely fastened!

- Before use, always check that the product is in proper working condition and correctly positioned - risk of injury!
- Do not attempt to repair or glue damaged components such as the webbing, toothed strap (b), or padding (a) and reuse them - risk of accidents!
- Do not lift the wheelchair above the **PELVI.LOC®** - risk of accident!

Restrictions on use / undesirable side effects

- Do not use the **PELVI.LOC®** if there are pressure sores, wounds, or other unhealed injuries in the area of the padding - risk of injury!
- Do not use the **PELVI.LOC®** in cases of osteoporosis, brittle bone disease, or severe circulatory disorders - risk of injury!
- Ensure proper functionality of the **PELVI.LOC®** after cleaning!
- A damaged **PELVI.LOC®** should be replaced immediately - risk of accident!



Caution:

- The **PELVI.LOC®** is not part of the wheelchair restraint system and does not serve to secure the occupant during transport in a vehicle for disabled persons.
- Neither the manufacturer nor the distributor can be held liable under product liability laws for damages or risks caused by unsupervised children in connection with this product!

Application

1. To use the **PELVI.LOC®**, place the padding (a) over the instep



Caution: Ensure that no objects are underneath the padding! This prevents painful pressure points.

2. Next, thread the toothed strap (b) through the respective ratchet. To do this, operate the corresponding ratchet buckle (e).
 3. Then tighten the toothed strap (b) until there is enough room to fit a flat hand between the ankle and the pad.
 4. Make sure that the snaps (e) engage with the teeth of the toothed strap!
- Perform a tensile test!

If the positioning system loosens unintentionally during use of the **PELVI.LOC®**, this can be rectified by re-tensioning.

To open the **PELVI.LOC®**, press the catch and pull out the toothed strap.

- Make sure that the padding does not get caught between the spokes of the drive wheel!

Cleaning and disinfection

- Machine washable up to 40 °C.
- Chemothermal disinfection possible by adding ELTRA 40 to the wash cycle
- Do not spin
- Not suitable for tumble drying

Maintenance

The **PELVI.LOC®** should be regularly inspected for damage and functionality.

Repairs

A defective **PELVI.LOC®** should be replaced immediately.

Disposal

- The device packaging can be recycled.
- The metal components can be recycled as scrap metal.
- Plastic components can be recycled as plastic waste.
- Disposal must comply with applicable national legal regulations.
- Please contact your local city or municipal administration for information on waste disposal companies.

Technical data

	S	M	L	XL
Circumference (cm)	17 - 23	20 - 26	24 - 30	26-34
Toothed strap	R-19-VI-BL-210	R-19-VI-BL-275	R-19-VI-BL-300	R-19-VI-BL-300

latex free



Storage and transport conditions

	Long-term storage temperature range: -10 °C to +40 °C. Short-term temperature fluctuations, such as during transport, from -20 °C to +60 °C are well tolerated by the material and do not affect the product's safety or performance.
	Protect from moisture during storage and transport

Warranty

We provide a warranty for this product in accordance with statutory regulations. This warranty covers material and manufacturing defects. Excluded from the warranty are wear parts and components subject to normal wear and tear, as well as damage resulting from excessive use, improper application, forceful damage, or unauthorised modifications/repairs. In the event of a warranty claim, please contact the specialist dealer from whom you purchased this product. For product liability reasons, repair, maintenance, and servicing must only be carried out by authorised specialist dealers using original spare parts



Manufacturer:

A2J GmbH
Am Fuchsberg 13
D-87452 Altusried
Tel: +49 (08373) 987269
Fax: +49 (08373) 987276
email: info@a2j-intl.com
Internet: www.a2j-intl.com



This product complies with the essential requirements of EU Regulation 2017/745 on medical devices – Medical Device Regulation (MDR)

As of 01.01.2025

Subject to technical modifications