



## **PELVI.LOC® PL-RR - XS - S - M - L - XL**



For the safe use of this product, please read and follow these assembly and usage instructions before first use



### **Area of application**

The **PELVI.LOC®** PL-RR Instep Strap is designed for individuals with limited mobility, specifically for foot positioning on foot shells and footrests, preventing slipping forward.

### **Intended patient group:**

- Children, adolescents, and adults
- Gender-neutral

### **Receipt control**

Unpack the product completely and check the shipment for completeness and transport damage. If you unexpectedly find any defects in the shipment:

1. Immediately report the issue to the carrier
2. Have the damage or defect documented
3. Inform us or your specialist dealer without delay

### **Scope of delivery**

- 1) 1 x **PELVI.LOC®** PL-RR Instep Strap -XS, -S, -M, -L, -XL pre-assembled (pair)
- 2) 1 x Mounting set for footplates
- 3) 1 x Information for online download of the assembly and usage instructions

### **Combination with medical devices**

Suitable for assistive devices designed for the use of instep straps

## Assembly



Assembly should be carried out by a person with appropriate expertise

1. Place the padding (a) in the instep area of the foot. The toothed strap (b) is attached to the footplate using the mounting set
2. Adjust the distance of the padding using the mounting set (d).
3. Tighten the instep strap by operating the ratchet buckles (e).
4. Release the instep strap by pressing the button (f).
5. Tension the toothed strap (b) so that the foot is pressed onto the footplate
6. If necessary, the ends of the toothed strap can be trimmed to a rounded shape. Deburr the trimmed ends of the toothed strap if needed – risk of injury!
7. Check the secure fit and locking on both sides of the toothed strap!
8. Check the secure fit and locking on both sides of the toothed strap!



## Instructions for use



## Safety

## instructions

Despite its robust construction, components such as the mounting set, toothed straps, and padding are subject to wear and tear. Therefore, it is recommended to thoroughly inspect the **PELVI.LOC®** every 3 months. Ensure that rivet and screw connections are securely fastened!

- Before use, always check that the product is in proper working condition and correctly positioned - risk of injury!
- Do not attempt to repair or glue damaged components such as the webbing, toothed strap (b), or padding (a) and reuse them - risk of accidents!
- Do not lift the wheelchair above the **PELVI.LOC®** - risk of accident!

## Restrictions on use / undesirable side effects

- Do not use the **PELVI.LOC®** if there are pressure sores, wounds, or other unhealed injuries in the area of the padding - risk of injury!
- Do not use the **PELVI.LOC®** in cases of osteoporosis, brittle bone disease, or severe circulatory disorders - risk of injury!
- Ensure proper functionality of the **PELVI.LOC®** after cleaning!
- A damaged **PELVI.LOC®** should be replaced immediately - risk of accident!



## Caution:

- The **PELVI.LOC®** is not part of the wheelchair restraint system and does not serve to secure the occupant during transport in a vehicle for disabled persons.
- Neither the manufacturer nor the distributor can be held liable under product liability laws for damages or risks caused by unsupervised children in connection with this product!

## Application

1. To use the **PELVI.LOC®**, place the padding (a) over the instep



**Caution:** Ensure that no objects are underneath the padding! This prevents painful pressure points.

2. Next, thread the toothed strap (b) through the respective ratchet. To do this, operate the corresponding ratchet buckle (e).
3. Then, tighten the toothed strap (b) just enough so that the foot is pressed onto the footplate (light contact)

If the positioning system loosens unintentionally during use of the **PELVI.LOC®**, this can be rectified by re-tensioning.

**To open the **PELVI.LOC®****, press the release lever (f) and pull out the toothed strap.

- Make sure that the padding does not get caught between the spokes of the drive wheel!

## Cleaning and disinfection

- Machine washable up to 40 °C.
- Chemothermal disinfection possible by adding ELTRA 40 to the wash cycle
- Do not spin
- Not suitable for tumble drying

## Maintenance

The **PELVI.LOC®** should be regularly inspected for damage and functionality.

## Repairs



A defective **PELVI.LOC®** should be replaced immediately.

## Disposal

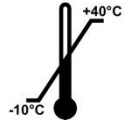

- The device packaging can be recycled.
- The metal components can be recycled as scrap metal.
- Plastic components can be recycled as plastic waste.
- Disposal must comply with applicable national legal regulations.
- Please contact your local city or municipal administration for information on waste disposal companies.

## Technical data

PL-RR	XS	S	M	L	XL
<b>Padding size (cm)</b>	17.5 x 5	17.5 x 5	25.5 x 8	25.5 x 8	24 x 7
<b>Total length (cm) min</b>	19	20	24	24	29
<b>Total length (cm) max</b>	27	27	31	36	51
<b>Toothed strap</b>	R-14-GO-110	R-24-BL-170	R-24-BL-195	R-24-BL-225	R-24-BL-225
<b>Buckle</b>	B-14-GO-BL	B-24-LU-JU	B-24-LU-JU	B-24-LU-JU	B-24-LU-JU

PL-RR-GO	S	M	<p>Product is latex free</p>  
<b>Padding size (cm)</b>	17.5 x 5	25.5 x 8	
<b>Total length (cm) min</b>	20	24	
<b>Total length (cm) max</b>	27	31	
<b>Toothed strap</b>	R-19-GO-BL-350	R-19-GO-BL-350	
<b>Buckle</b>	B-19-GO-BL	B-19-GO-BL	

## Storage and transport conditions

	<p>Long-term storage temperature range: -10 °C to +40 °C.</p> <p>Short-term temperature fluctuations, such as during transport, from -20 °C to +60 °C are well tolerated by the material and do not affect the product's safety or performance.</p>
	<p>Protect from moisture during storage and transport</p>

## Warranty

We provide a warranty for this product in accordance with statutory regulations. This warranty covers material and manufacturing defects. Excluded from the warranty are wear parts and components subject to normal wear and tear, as well as damage resulting from excessive use, improper application, forceful damage, or unauthorised modifications/repairs. In the event of a warranty claim, please contact the specialist dealer from whom you purchased this product. For product liability reasons, repair, maintenance, and servicing must only be carried out by authorised specialist dealers using original spare parts.



**Manufacturer:**

A2J GmbH

Am Fuchsberg 13

D-87452 Altusried

Fon: +49 (08373) 987269

Fax: +49 (08373) 987276

eMail: [info@a2j-intl.com](mailto:info@a2j-intl.com)

Internet: [www.a2j-intl.com](http://www.a2j-intl.com)



This product complies with the essential requirements of the EU Regulation 2017/745 on medical devices – Medical Device Regulation (MDR)