



PELVI.LOC®

Technology that connects

Installation instructions

Upper body - positioning to measure

1.

Carry the strap between the seat cushion and the back cushion and close it on the back with the safety lock and tighten the strap



2.

Cover the seat pants with the enclosed plastic hose (which prevents the hook tape from getting caught in the velcro surface of the seat cushion) between the seat cushion and the back padding



3.

Pull the seat pants slightly up to the waist in the crotch



4.

Position shoulder straps on the seat pants



5.

Pull the right side of the abdominal belt to the left, then place the left side of the abdominal belt over to the right and close it with the safety buckle

**6.**

Pull out the plastic sleeve of the seat pants to the rear

**7.**

Attach seated trousers with Velcro surface to the Velcro surface of the center belt

**Note:**

- only with fine wash in the washing machine at 30 ° C washing
- do not spin
- do not put in the dryer
- Always secure OPS positioned person with car 3 point belt