

Assembly Instructions Pelvi-Loc 2DS / 3DS / 2DAS / 3DAS

Picture 1: Lead the webbing around the tube

Picture 2: Drill through the middle of the webbing with a graver or soldering iron

Picture 3: M6 screw with washer through the belt strap Screw and washer and nut tighten

Picture 4: Tighten the washer with the nut on the back of the belt

Picture 1




Picture 2



Picture 3



Picture 4

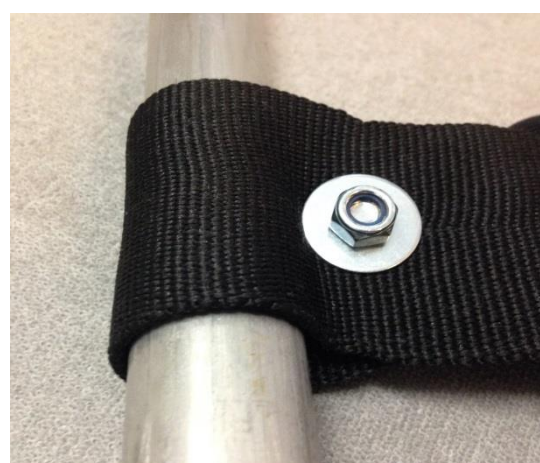

The webbing can also be punched several times, keep a distance of at least 2 cm from hole to hole.