




PELVI.LOC PL - 3DWP -L

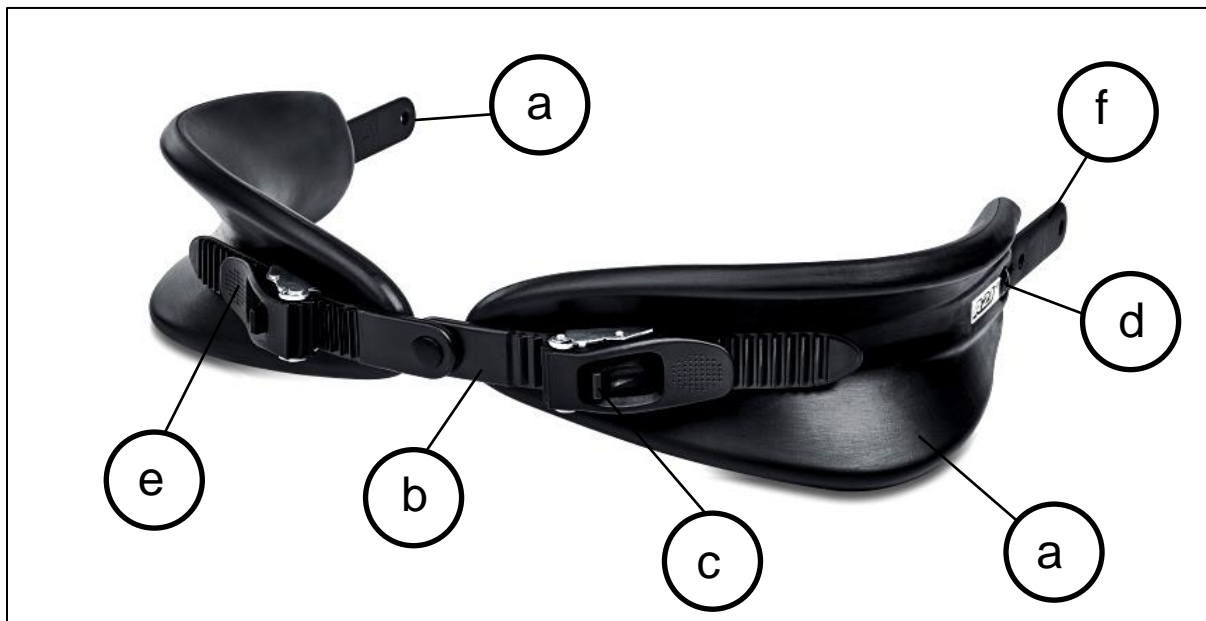

For the safe use of this product, read and observe these assembly instructions and the manual before the first use.



Field of Application

This Pelvi.Loc PL-3DWP Waterproof is intended for improved positioning and pelvis retraction of people with restricted mobility on a seating surface, e.g. wheelchair, seat shell, therapy chair, rehab buggy or piece of sports equipment.

Intended Patient Group:

- Adolescents and adults
- Sex: female and male

Receipt Control

Completely unpack the product and check the delivery for completeness and transport damage. Should you – against expectation – find a defect in the delivered product, then:

1. Immediately lodge a complaint with the freight carrier
2. Have the damage/defect recorded
3. Notify us or your retailer immediately

Scope of Delivery

- 1) 1 x PELVI.LOC PL-3DWP pre-assembled
- 2) 1 x Informationen zum online Download der Montage- und Gebrauchsanweisung / Hinweis

Combination of medical devices

Suitable for therapeutic appliances which are intended for the use of a pelvis retraction belt

Assembly



Assembly should be carried out by a person with specialist knowledge.

1. Place the paddings (a) respectively on the pelvic area/antecubital fold of the hip. – With the ratchet (b) facing upwards.
 - Afterwards, the logo on the padding should be readable from the front!
2. Set the spacing of the paddings via the ratchet (b) by operating the pressure buckles (e).
 - The hinge of the ratchet should then be centred between the paddings.
 - Pull out the ratchet no further than to the point, where two teeth are still free behind each catch (c).
1. After that, attach the hole straps/webbing (f) to the frame tube of the wheelchair (observe assembly notes)
2. Tighten the ratchet (b) on both sides equally and to a degree that still allows enough space for a flat hand between the thigh and the padding.
3. Both ends of the ratchet can be cut to length in a semicircle, if required. Deburr the ends of the cut ratchet, if necessary. –Danger of injury!
4. Check the firm fit and locking on both sides of the ratchet!



Manual



Safety Notes

In spite of their stable construction, components like the hole straps, webbings, ratchets and paddings are subject to wear and tear. It is therefore recommended to perform a thorough inspection of the PELVI.LOC PL-3DWP every 6 months. It is thereby important to make sure the rivet and screw connections are firmly secure!

- Before using the product, ensure that it is in a functional condition and that it fits correctly. – Danger of accidents!
- Do not patch up and/or glue up and reuse damaged components such as webbing/hole strap (f), ratchet (b) or padding (a). –Danger of accidents!
- Do not sit on the paddings – danger of bruising, especially by the catches (c)!
- Do not lift the wheelchair by the PELVI.LOC PL-3DWP. Danger of accidents!
- Be careful that your hands do not get caught between the hole strap and the frame or seat tube. Danger of crushing!

Restrictions on Use/Undesirable Side Effects

- Do not use the PELVI.LOC PL-3DWP, if there are any bruises, wounds or other not well-healed injuries in the area where the paddings will be. –Danger of injury!
- Do not use the PELVI.LOC PL-3DWP in case of osteoporosis, brittle bone disease or severe circulatory disorders. –Danger of injury!
- After cleaning, ensure the correct functioning of the PELVI.LOC PL-3DWP!
- A damaged PELVI.LOC PL-3DWP must be replaced immediately. –Danger of accidents!



Attention:

The PELVI.LOC PL-3DWP is not part of the restraint system for the wheelchair and/or user during transport in a vehicle for the transport of disabled persons.

- Neither the manufacturer nor the distributor will be liable, in the sense of product liability, for damage or risks caused by unattended children with regard to the product.

Use

1. To use the PELVI.LOC PL-3DWP, place the paddings (a) on the thighs.



Attention:

2. Make sure that there are no items under the padding! This will prevent painful bruises.
2. After that, guide the ratchet (b) through the corresponding catch. For that, operate the corresponding pressure buckle (e).
3. Then, tighten the ratchet (b) to a degree that still allows enough space for a flat hand between the thigh and the padding.
4. Make sure that the catches (c) engage with the teeth of the ratchet!
 - o Check the tension!

If the positioning system becomes loose unintentionally while the PL-3DWP is being used, this can be remedied by retightening the system.

To open the PELVI.LOC PL-3DWP, operate the catch (c) and pull the ratchet out.

- Make sure that the paddings do not get caught between the stokes of the driving wheel!

Cleaning and Disinfection

- Wipe disinfection using ISOPROPYL ALCOHOL 70%
- "Do not use any solvents such as acetone or benzine!"

Maintenance

Inspect the PELVI.LOC PL-3DWP for damage and check the functioning on a regular basis.

Repair

Immediately replace defective PELVI.LOC systems.

Disposal

- The packaging is recyclable.
- The metal parts can be disposed of through scrap metal recycling.
- Plastic parts are suitable for plastic recycling.
- The disposal must be made according to national statutory provisions.
- Please inquire with your city or municipal administration about local disposal services.

Technical Data

Seat width:	L	36 to 46 cm
Padding size:	L	27.0 x 13.5 cm
Weight:	L	Approximately 320 g
Predetermined breaking point:	L	At approximately 80 kg

Ratchet size: M-L	2.4 x 0.2 x 17.0 cm
Pressure buckle	B-24-V2A-BL
EVA paddings, closed-pore material	

Product is latex-free



Warranty

We provide a warranty for the product in accordance with the legal provisions. This warranty includes material and processing defects. This does not include wear parts and components/assemblies that are subject to intended wear and tear as well as damage that is the result of excessive strain, "improper use", forcible damage or unauthorised modification/repair. If you wish to make a warranty claim, please inquire with the retailer you purchased the product from. For product liability reasons, repairs and maintenance work must only be conducted by authorised speciality retailers, using original spare parts.



Manufacturer:

A2J GmbH
Am Fuchsberg 13
87452 Altusried, Germany
Phone: +49 (08373) 987269
Fax: +49 (08373) 987276
email: info@a2j-intl.com
Internet: www.a2j-intl.com



This product complies with the fundamental requirements of EC Directive 93/42/ECC concerning medical devices

As of 01/11/2018 subject to technical modifications